



### Bag Spike (SH-10) and Bag Spike Adapter (SH3034) for use with dedicated pump set

#### 1. Receive Prepared IV Bag



- › Prepared hazardous drug will come from pharmacy spike with a bag spike (SH-10).

#### 2. Prime IV Tubing



- › Prime administration set per facility protocol.
- › Open clamp and squeeze drip chamber.

#### 3. Insert IV Tubing



- › Insert spike of pre primed IV tubing set into distal end of the bag spike adapter (SH3034).
- › Make sure spike is fully inserted to the bevel on the bag spike.

#### 4. Attach Spiros to End of IV Tubing



- › Attach Spiros to end of IV tubing.

#### 5. Attach Spiros to Clave Bag Spike



- › Luer lock Spiros on bag spike adapter (SH3034) to bag spike (SH-10) and administer hazardous drug.

#### 6. Administer



- › Connect Spiros on end of IV tubing to patient end.