

IMPROSAFE® BloodCollectionSet provides



Benefit

- Ultra sharp needle makes venipuncture smooth and gentle
- Large, flexible wings with ergonomic design provide comfortable grip
- Blood flashback to be observed as sign of venipuncture success

Convenience

- Pre-attached holder eliminates steps in preparation process before venipuncture
- Flexible tubing gives convenient operation and reduces patient discomfort due to changing a succeeding tube
- No need to prepare holders separately and check the quantity equivalence of blood collection sets

Safety

- Safety shield protects healthcare staff from needle exposure and needlestick injuries
- Needle is completely and securely locked after use, preventing reuse
- Pre-attached holders prevent risk of accidental needlestick injuries

* The protective shield is designed to help your institution meet current Federal OSHA safety standards for engineering controls 1999 Federal OSHA Compliance. The pre-attached holder is designed to help your institution comply with OSHA single-use holder directive OSHA SHIB 10-15-03, by reducing the risk of holder reuse.

HUMAN HEALTH WE CARE

Specialize in Specimen Processing

Specialize in Clinical Laboratory

Specialize in Clinical Nursing



IMPROVE MEDICAL


🏠 No. 102, Kaiyuan Ave. Science City, 510530, Guangzhou, China
 ☎ 86-20-32312610/1/2/3/4/5/6/7
 📠 86-20-32312667/2611
 🌐 www.improve-med.com
 ✉ info@improve-medical.com

FCO: 20140718



IMPROSAFE® BloodCollectionSet

Available with Pre-attached Holder

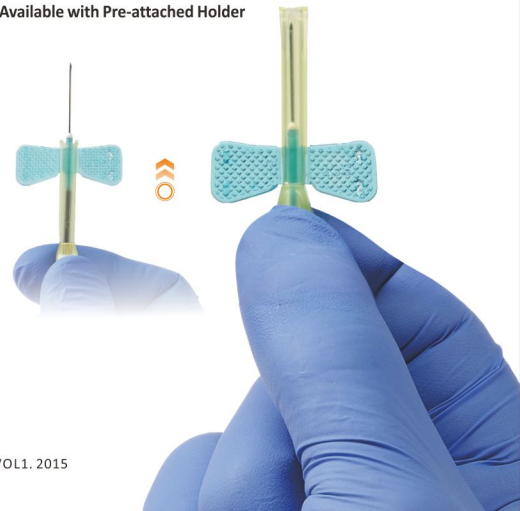

QT-20-08-002 A/0

VOL1. 2015

EASY TO USE One-handed Technique




1 Withdraw safety blood collection set by holding the yellow safety shield grip area with the thumb and index finger



2 Then while holding the tubing securely with the other 3 fingers



3 Push the yellow safety shield forward with thumb and index finger

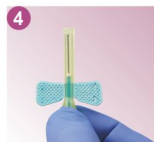

4 Until the needle is completely covered and a "click" is heard, indicating that the safety shield is locked in place over the needle tip



5 Discard immediately into an approved sharps container

EASY TO USE Two-handed Technique



1 Withdraw safety blood collection set by holding the yellow safety shield grip area with the thumb and index finger



2 With the opposite hand, hold the tubing between thumb and index finger.



3 Push the yellow shield forward until the needle is completely covered and a "click" is heard, indicating that the safety shield is locked in place over the needle tip.



4 Discard immediately into an approved sharps container.

Specification

Product Code	Needle/Gauge x Needle Length x Tubing Length	Wing Color	QTY/BX x BX/CS (PE Bag)	QTY/BX x BX/CS (Blister Pkg)
IMPROSAFE® Blood Collection Set				
596121300	21G x 3/4" x 12"	Green	100 x 20	50 x 20
596121190	21G x 3/4" x 7"	Green	100 x 20	50 x 20
596123300	23G x 3/4" x 12"	Blue	100 x 20	50 x 20
596123190	23G x 3/4" x 7"	Blue	100 x 20	50 x 20
596125300	25G x 3/4" x 12"	Orange	100 x 20	50 x 20
596125190	25G x 3/4" x 7"	Orange	100 x 20	50 x 20
IMPROSAFE® Blood Collection Set with Pre-attached Holder				
596121302	21G x 3/4" x 12"	Green	50 x 10	
596121192	21G x 3/4" x 7"	Green	50 x 10	
596123302	23G x 3/4" x 12"	Blue	50 x 10	
596123192	23G x 3/4" x 7"	Blue	50 x 10	
596125302	25G x 3/4" x 12"	Orange	50 x 10	
596125192	25G x 3/4" x 7"	Orange	50 x 10	