

# Ketone Testing Made Easy!



## Positive Results = Purple Color<sup>1</sup>

The test is positive if a purple color develops on the tablet, read urine at 30 seconds, serum & plasma at 2 minutes, whole blood at 10 minutes.

## Negative Results = No Purple Color

The test is negative if no purple color develops on the tablet, read urine at 30 seconds, serum & plasma at 2 minutes, whole blood at 10 minutes

## AimTab™ Ketone Tablets

Same test procedure as Acetest®, a registered trademark used by Bayer HealthCare LLC.



## CLIA WAIVED For Urine Testing

- AimTab™ Ketone Tablets, Ref #13100 100/bottle.
- Quick & easy ketone results for testing urine, serum, and whole blood.
- Detects as little as 5 mg/dL of acetoacetic acid in urine.
- Specific for acetoacetic acid and acetone and will not react with betahydroxybutyric acid.
- CLIA moderately complex when testing serum, plasma or whole blood.
- CAP / CLSI formatted procedures available via fax or email - just add your name!
- Store at room temperature.

## Related Products:

- UA Liquid Controls, w/ hCG, Ref #51116  
2 x 5mL (1mL level1, 1mL level 2)  
Used for Ketone control.  
Store at 2° to 8°C.

<sup>1</sup> See Package Insert for complete procedure.

©2015 Germaine Laboratories, Inc., CSM 0800-41, Rev. 10-15

Distributed by:

Tests you Want,  
Accuracy you Need!



www.germainelabs.com

800.854.8446

210.692.4192

210.692.4198 Fax

