FLEX-SUPPORT®



Quality Medical Products Since 1965

Industrial Safety Products

FRANK STUBBS CO., INC. 2100-B EASTMAN AVENUE OXNARD, CA 93030 (800) 223-1713 FAX (805) 278-6609 www.fstubbs.com

Catalog Contents Flex Support®

Product

- 9" Black Industrial Back Support Without Suspenders (#R3025B)
- 9" Black Industrial Back Support With Suspenders (#R3026B)
- 9" Black Industrial Back Support Without Suspenders, "Cool Belt" (#R3035)
- 9" Black Industrial Back Support With Suspenders, "Cool Belt" (#R3036)
- 7" Black Industrial Back Support Without Suspenders, "Cool Belt" (#3042B)
- 7" Black Industrial Back Support With Suspenders, "Cool Belt" (#3043B)
- 7" Black Industrial Back Support Without Suspenders (#3045B)
- 7" Black Industrial Back Support With Suspenders (#3046B)

Tennis Elbow Support (#3102)

Black Neoprene Wrist Wrap (#3101)

CTD Wrist Stabilizer (#2137B)

Industrial Back Supports

These supports are designed to help reduce the potential of work related injuries.

These products will help the employer reduce their work related costs

(i.e. Worker's compensation) while increasing productivity and performance.

9" Black Industrial Back Support Without Suspenders (#R3025B) 9" Black Industrial Back Support With Suspenders (#R3026B)



Support is constructed of surgical elastic back panels with 4 flexible stays
Side panels are lined with non-stretch foam and contact closure provides secure abdominal support
Double pull elastic tension straps with contact closure

Available in: YS S P XI and XXI

Available in: XS, S, R, XL, and XXL

Latex free

Indications: These back supports are designed to help reduce potential work related injuries.

Application Instructions: Detach double pull straps and align back panel at lower back. Grasp ends of the side panels and stretch them right over left. Reach for the double pulls, stretch and secure them. Be sure belt covers the abdomen, as the belt should provide abdominal support. With suspenders, adjust strap length so that straps remain firmly but comfortably on shoulders.

Sizing: Take measurement around mid-section at belly button.

Sizes: XS (20"-26"), S (26"-36"), M (36"-46"), R (36"-46"), XL (46"-60"), and XXL (58"-70")

9" Black Industrial Back Support Without Suspenders, "Cool Belt" (#R3035)

9" Black Industrial Back Support With Suspenders, "Cool Belt" (#R3036)



Constructed of vented surgical elastic back panels with 4 flexible stays Non-stretch side panels with contact closure for abdominal support Double pull elastic tension straps with contact closure Available in: XS, S, R, XL, and XXL Latex free

Indications: These back supports are designed to help reduce potential work related injuries.

Application Instructions: Detach double pull straps and align back panel at lower back. Grasp ends of the side panels and stretch them right over left. Reach for the double pulls, stretch and secure them. Be sure belt covers the abdomen, as the belt should provide abdominal support. With suspenders, adjust strap length so that straps remain firmly but comfortably on shoulders.

Sizing: Take measurement around mid-section at belly button.

Sizes: XS (20"-26"), S (26"-36"), M (36"-46"), R (36"-46"), XL (46"-60"), and XXL (58"-70")

Industrial Back Supports

7" Black Industrial Back Support Without Suspenders, "Cool Belt" (#3042B)

7" Black Industrial Back Support With Suspenders, "Cool Belt" (#3043B)



Constructed of vented surgical elastic back panels with 4 flexible stays Non-stretch side panels with contact closure for abdominal support Double pull elastic tension straps with contact closure Available in: XS, S, R, and XL Latex free

Indications: These back supports are designed to help reduce potential work related injuries.

Application Instructions: Detach double pull straps and align back panel at lower back. Grasp ends of the side panels and stretch them right over left. Reach for the double pulls, stretch and secure them. Be sure belt covers the abdomen, as the belt should provide abdominal support. With suspenders, adjust strap length so that straps remain firmly but comfortably on shoulders.

Sizing: Take measurement around mid-section at belly button.

Sizes: XS (20"-26"), S (26"-36"), M (36"-46"), R (36"-46"), and XL (46"-60")

7" Black Industrial Back Support Without Suspenders, (#3045B)
7" Black Industrial Back Support With Suspenders, (#3046B)



Constructed of surgical elastic back panels with 4 flexible stays Non-stretch side panels with contact closure for abdominal support Double pull elastic tension straps with contact closure Available in: XS, S, R, and XL Latex free

Indications: These back supports are designed to help reduce potential work related injuries.

Application Instructions: Detach double pull straps and align back panel at lower back. Grasp ends of the side panels and stretch them right over left. Reach for the double pulls, stretch and secure them. Be sure belt covers the abdomen, as the belt should provide abdominal support. With suspenders, adjust strap length so that straps remain firmly but comfortably on shoulders.

Sizing: Take measurement around mid-section at belly button.

Sizes: XS (20"-26"), S (26"-36"), M (36"-46"), R (36"-46"), and XL (46"-60")

Industrial Elbow Support

These supports are designed to help reduce the potential of work related injuries.

These products will help the employer reduce their work related costs

(i.e. worker's compensation) while increasing productivity and performance.





2" black webbing material with loop and lock contact closure One size Latex free

Indications: Elbow supports provide compression to support stretched or weakened muscles, minor sprains and strains, and helps decrease symptoms of tendinitis.

Application Instructions: Slide support and position at thickest part of the forearm. Loop strap through the metal ring, apply needed compression and secure with contact closure.

Sizes: One size

Industrial Wrist Support

These supports are designed to help reduce the potential of work related injuries.

These products will help the employer reduce their work related costs

(i.e. worker's compensation) while increasing productivity and performance.

Black Neoprene Wrist Wrap (#3101)



Wrap around style with thumb hole Constructed of 1/8" neoprene covered with 2-sided nylon and contact closure One size Latex free

Indications: Provides support for the wrist to help prevent injuries.

Application Instructions: Place thumb through thumb hole. Stretch short end of support around break of wrist. Stretch long end, wrap around and attach first contact closure. Continue wrapping and attach second contact closure.

Sizes: One size

CTD Wrist Stabilizer (#2137B)



All elastic support with Orthowick lining to eliminate moisture Tension strap wraps around wrist for stability Support can accommodate either right or left hand Available in: S, M, L, and XL Latex free

Indications: Promotes neutral position of wrist when not in use. Ideal support for carpal tunnel syndrome.

Application Instructions: Slide hand into support with thumb through the thumb hole. Tighten tension strap.

Sizing: Measure width of palm at base of thumb.

Sizes: S (2 $\frac{1}{2}$ "-3"), M (3"-3 $\frac{1}{2}$ "), L (3 $\frac{1}{2}$ "-4"), and XL (4"-4 $\frac{1}{2}$ ")